

HCV Control Log for (month) _____ / (year) _____

Date	QC Code	Test Device		Control Kit			Negative Control Result			HCV Ab+ Control Result			Results Acceptable?		Initials
		Lot #	Exp date	Lot #	Exp date	Date opened	Start time/temp	End time	Circle Result	Start time/temp	End time	Circle Result	Y	N	
									NRI			NRI	Y	N	
									NRI			NRI	Y	N	
									NRI			NRI	Y	N	
									NRI			NRI	Y	N	
									NRI			NRI	Y	N	
									NRI			NRI	Y	N	
									NRI			NRI	Y	N	
Quality Control Code (reason for running controls) <ol style="list-style-type: none"> 1. New setting 2. New operator 3. New test kit lot 4. New test kit shipment 5. Environmental change – temp outside range 6. Environmental change – temp outside range in <u>test</u> area 7. Environmental change – low lighting 8. Scheduled, periodic test 9. Other – document on back of this form 						N – Non-Reactive R - Reactive I - Invalid Acceptable Results Negative and positive HCV controls must yield correct results. If either one yields an incorrect result, the results are <u>unacceptable</u> . DO NOT conduct any tests until the problem is resolved. Document problem and corrective action on the back of this form.									

Initial Review _____ / _____ / _____
 Signature _____ Date _____

Final Review _____ / _____ / _____
 Signature _____ Date _____

Must be reviewed by someone other than the nurse or counselor who ran the controls.